

# NC-9000DRX with a Notification Display Monitor:

## Quick Start Manual


v0.0.2



Excel Engineering Co.,LTD.  
株式会社エクセルエンジニアリング

# NC-9000DRX: Schematic Diagram

This system is the on-premises type monitoring system. When the user makes a call, an alert message will be received on the PC.

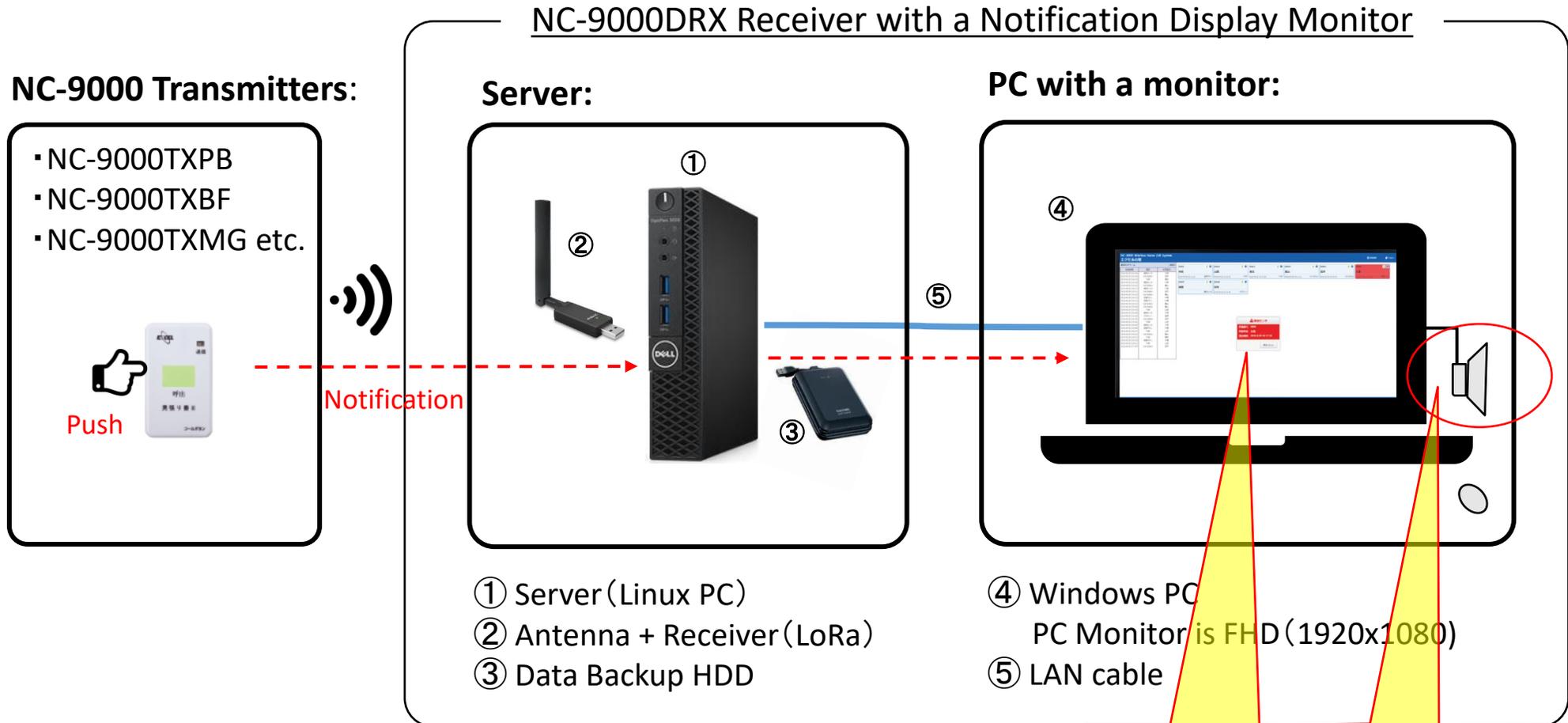

\*The red lines represent how the alert signals are routed.

Alert information will be displayed on the monitor and there will be an alert sound

# NC-9000DRX: Main Menu Screen

## Main Menu Screen:

The main menu screen displays notification history, registered device information, and received alert messages.

The screenshot shows the main menu of the NC-9000 Wireless Nurse Call System. The interface includes a header with the system name and status, a 'Recent alarm' table, a grid of registered devices, and a central alert popup. Callouts 1, 2, and 3 highlight the notification history, device registration, and language settings respectively. A yellow callout explains that alert-receiving devices turn red, and another explains that a popup message is shown upon receiving an alert.

**Recent alarm**

Reception time	Type	User name
2019.05.09 [18:20]	CallButton	KIKUCHI
2019.05.09 [18:20]	Door	YASUOKA
2019.05.09 [18:20]	Door	YASUOKA
2019.05.09 [18:20]	CallButton	MAEDA
2019.05.09 [18:20]	CallButton	KIKUCHI
2019.05.09 [18:19]	CallButton	DARVISH
2019.05.09 [18:19]	CallButton	DARVISH
2019.05.09 [18:19]	Chair	TANAKA
2019.05.09 [18:19]	CallButton	KIKUCHI
2019.05.09 [18:18]	CallButton	KIKUCHI
2019.05.09 [18:18]	CallButton	MAEDA
2019.05.09 [18:17]	CallButton	KIKUCHI
2019.05.09 [18:16]	CallButton	KIKUCHI
2019.05.06 [15:06]	CallButton	DARVISH
2019.05.06 [15:05]	Door	YASUOKA
2019.05.06 [15:05]	CallButton	KIKUCHI
2019.05.04 [04:17]	Chair	TANAKA
2019.05.04 [04:17]	Door	YASUOKA
2019.05.04 [04:17]	Chair	TANAKA
2019.05.04 [04:17]	Chair	TANAKA
2019.05.04 [04:17]	Chair	TANAKA

**Registered devices**

Control number	User name	Reception time	Location
001	HIRANO	2019.05.04 04:04:25	Floor
101	YASUOKA	2019.05.09 18:20:26	Door
202	KIKUCHI	2019.05.09 18:20:04	CallButton
303	MAEDA	2019.05.09 18:20:11	CallButton
999	OHTANI	2019.01.15 00:28:22	Bench
303	! ICHIRO	2019.04.11 13:12:53	Bed
999	GUEST	2019.04.26 11:16:02	none

**Alert Popup**

**CallButton**

Control number: 202  
User name: KIKUCHI  
Reception 2019.5.9 18:18:34 time

confirmed

**Callouts:**

- ① This section displays received alert history
- ② This is where you can register a new transmitter information
- ③ Change the display language to Japanese or English

**Alert Message Callouts:**

- When an alert is received a registered device on the screen, it will turn red to inform you which device generated the alert
- A popup message will be displayed when an alert is received


**Bottom Labels:**

- Notification history display area
- Registered devices information display area
- A received alert message




# NC-9000DRX: How to register a transmitter

## Sensor registration screen:

This is where sensors ( including transmitters ) are registered. Registered devices will be displayed.

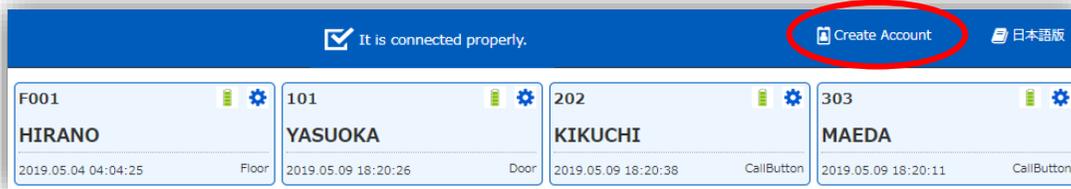


Click "Create Account" icon

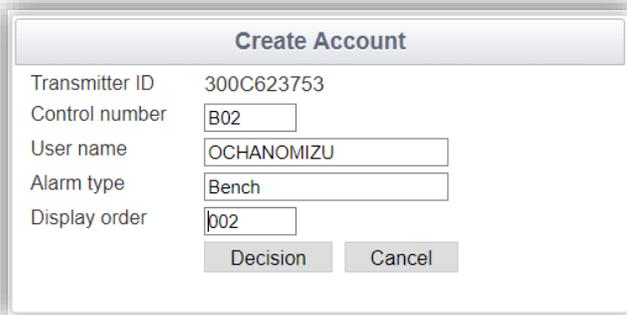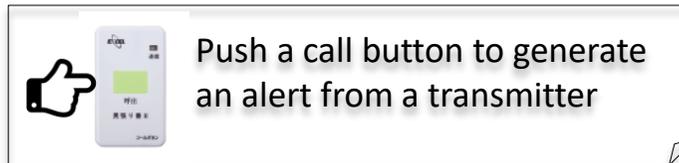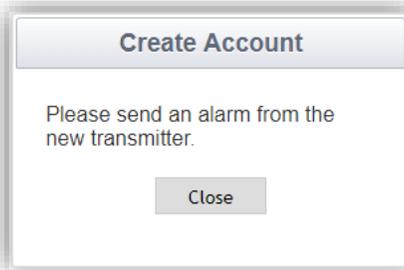

Enter desired information

Complete registration

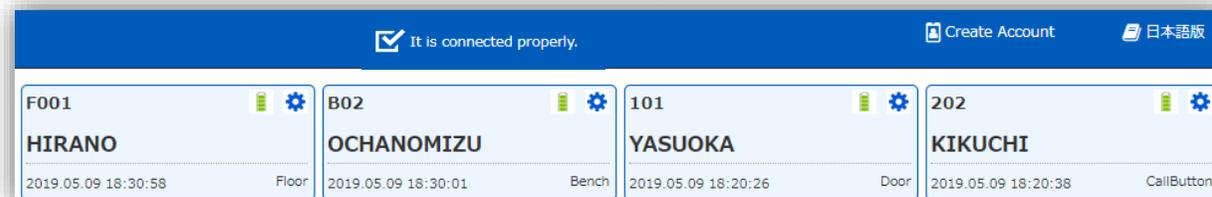

# NC-9000DRX: Notification History Screen

## Notification History Screen:

Notification history is displayed here. You can search for a specific event by entering dates and admin ID number. Displayed history can be exported as a CSV file for further analyses and record keeping.

NC-9000 Wireless Nurse Call System  
EXCEL

It is connected properly. Create Account 日本語版

Alarm history Back to top

display period / / ~ / / Search clear

Control number


Transmitter ID

Reception time	Alarm type	Control number	User name	Transmitter ID
2019.05.09 [18:30:58]	Floor	F001	HIRANO	310281376E
2019.05.09 [18:30:58]	clear	F001	HIRANO	310281376E
2019.05.09 [18:30:01]	none	999	GUEST	300C623753
2019.05.09 [18:20:38]	CallButton	202	KIKUCHI	350B793771
2019.05.09 [18:20:26]	Door	101	YASUOKA	4141543120
2019.05.09 [18:20:19]	Door	101	YASUOKA	4141543120
2019.05.09 [18:20:11]	CallButton	303	MAEDA	350F7D3771
2019.05.09 [18:20:04]	CallButton	202	KIKUCHI	350B793771
2019.05.09 [18:19:52]	CallButton	205	DARVISH	
2019.05.09 [18:19:46]	CallButton	205	DARVISH	

Output CSV

Copyright 2018 EXCEL ENGINEERING Co., Ltd. All Rihts Reserved

A generated CSV file can be opened with Excel spreadsheet

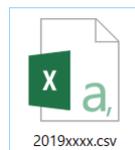

AutoSave On CSV\_read.xlsx - S

File Home Insert Page Layout Formulas Data Review View


A13 5/9/2019 6:19:16 PM

	A	B	C	D	E
1	Reception Time	Alarm type	Control No	User name	Transmitter ID
2	5/9/2019 18:30	Floor	F001	HIRANO	310281376E
3	5/9/2019 18:30	Floor	F001	HIRANO	310281376E
4	5/9/2019 18:30	none	999	GUEST	300C623753
5	5/9/2019 18:20	CallButton	202	KIKUCHI	350B793771
6	5/9/2019 18:20	Door	101	YASUOKA	4141543120
7	5/9/2019 18:20	Door	101	YASUOKA	4141543120

# NC-9000DRX:How to change the transmitter's registration information

## Transmitter Setup Screen:

You can change the transmitter's registration information and the order in which it is displayed on the main screen.

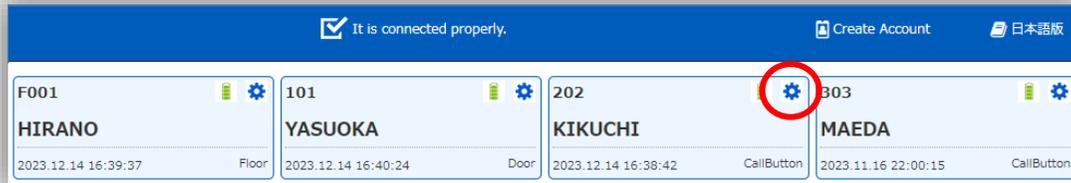

Click  icon

### Before change


Transmitter setting	
Transmitter ID	39157D375F
Control number	202
User name	KIKUCHI
Alarm type	CallButton
Display order	2
<input type="button" value="Decision"/> <input type="button" value="Cancel"/>	
To delete the display on this transmitter, enter the deletion code below and press the [ Delete ] button. ※Even if you delete it, the alarm history does not disappear.	
Delete Code	<input type="text"/> <input type="button" value="Delete"/>

### After change

Transmitter setting	
Transmitter ID	39157D375F
Control number	B001
User name	YAMADA
Alarm type	CallButton
Display order	1
<input type="button" value="Decision"/> <input type="button" value="Cancel"/>	
To delete the display on this transmitter, enter the deletion code below and press the [ Delete ] button. ※Even if you delete it, the alarm history does not disappear.	
Delete Code	<input type="text"/> <input type="button" value="Delete"/>


1. Change the input information  
In the "Display order" field, set the display order on the main screen
2. Click "Decision"

Change completed

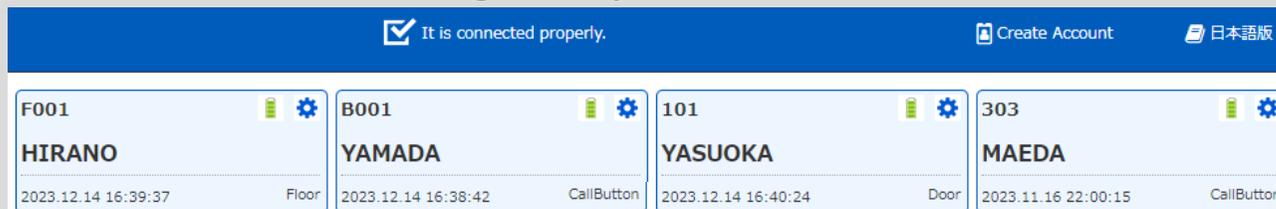

# NC-9000DRX: How to unregister a transmitter

Enter the deletion code in the transmitter setup screen to unregister the transmitter.

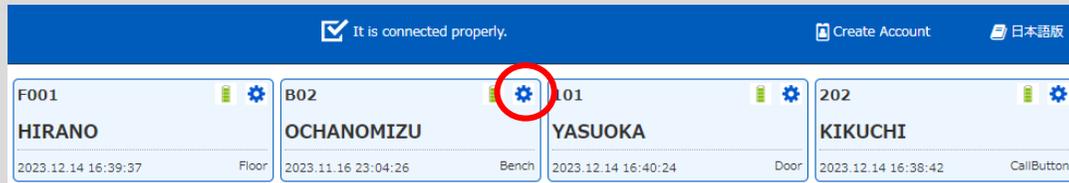

Click  icon

**Transmitter setting**

Transmitter ID 311657316A  
Control number   
User name   
Alarm type   
Display order

To delete the display on this transmitter, enter the deletion code below and press the [ Delete ] button.  
※Even if you delete it, the alarm history does not disappear.

Delete Code

Enter the following deletion code and click "Delete"  
Code: 9999

Unregistration completed

